

18 Gallon Sharps Take Back

Instructions

ARXG
TAKE BACK SOLUTIONS

Filling

1. Grab a red 18 gallon container, a lid, and an absorbent pouch. Keep your lid where you can find it later or hook it to the side of the container.
2. Place the included moisture-absorbant pouch in the bottom of the container.
3. You can now begin filling the 18 Gallon container with individual sharps containers.



Step: 1



Step: 2



Step: 3

IMPORTANT: DO NOT FILL WITH STRAY NEEDLES. DO NOT OVERFILL.

DISCLAIMER:
ONLY for disposal of materials free of pathogens and transferrable diseases. If waste was exposed, it MUST be disinfected prior to disposal. Drain all free liquids prior to disposal.

Shipping

1. Place the locking lid on top of the container so the locking tabs align with the holes surrounding the lid. Press down on the lid until all of the tabs have clicked into place.
2. Assemble the return box by gently pressing the far outer edges of the box inward, creating a 'tube', find the bottom of the box. Close the minor flaps first, carefully removing the adhesive backing.
3. Carefully align the Major Flaps over the box and firmly seal the flaps on top of the adhesive, making sure to align the edges to prevent a hole or gap in the bottom.
4. Flip the box the right side up and place the large grey zipper bag in the box, then put the sealed 18 gallon sharps container in the bag.
5. Seal the bag by zipping shut, removing as much air as possible. Remove the adhesive backing and roll the onto the opposite side of the bag so the front adheres to the back. This removes strain from the zipper enclosure and helps prevent tampering.
6. Close the flaps with adhesive first. Remove the adhesive backing, then fold the remaining flaps on top. **IMPORTANT: Ensure the flaps completely close and leave no gaps or visibility into the box.**



Step: 1



Step: 2



Step: 3



Step: 4



Step: 5



Step: 6



**MAX WEIGHT:
62 LBS (28.0 KG)**

The box is now ready for shipment!

ARXG
TAKE BACK SOLUTIONS

Questions? Contact American Rx Group
877-350-0693 • rxtakeback@amrxgroup.com