

28 GAL MAIL-IN TAKE BACK INSTRUCTIONS

FILLING

1. Grab the red 28 gallon container, dark liner bag, lid, and absorbent pouch. Keep your lid where you can find it later or hook it to the side of the container.
2. Place the dark liner bag inside the 28 gallon container. Place the moisture absorbent pouch in the bottom of the dark liner bag.
3. You can now begin filling the 28 gallon container.
Important: Do not fill with stray needles. Do not overfill.



Step 1



Step 2



Step 3

SHIPPING

1. Seal the dark liner bag by zipping it shut, removing as much air as possible. Remove the adhesive backing and roll onto the opposite side of the bag so the front adheres to the back.
2. Peel back the adhesive strip and place the lid on top. Press down the lid to ensure it's adhered into place.
3. Place the sharps container inside the return box.
4. Close the non-adhesive flaps first then fold the remaining flaps on top. **Important: Ensure the flaps completely close and leave no gaps or visibility into the box.**
5. Drop off at UPS office or hand to your local driver. The shipping label is pre-paid.



Step 1



Step 2



Step 3



Step 4



MAX WEIGHT: 50 LBS (22.68 KG)

Only for disposal of materials free of pathogens and transferable diseases. If waste was exposed, it must be disinfected prior to disposal. Drain all free liquids prior to disposal. DO NOT OVERFILL.

Track your product online at www.americanrxgroup.com/tracking/
Need help? Call (855) 269-1750 or visit www.americanrxgroup.com